Rx only

Made in Switzerland

ACCU-CHEK, ACCU-CHEK FLEXLINK and
ACCU-CHEK LINKASSIST are trademarks of Roche.

© 2018 Roche Diabetes Care

In US, distributed by:
Roche Diabetes Care, Inc.
Indianapolis, IN 46256, **USA**
Toll Free 1-800-280-7801

 Roche Diabetes Care GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany
www.accu-chek.com

ACCU-CHEK® LinkAssist

Instructions for use

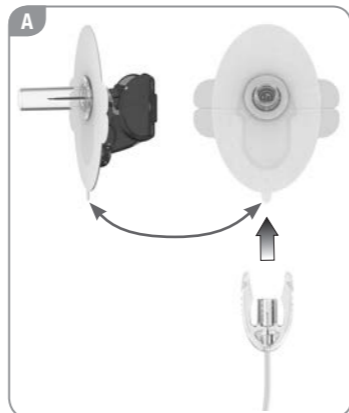
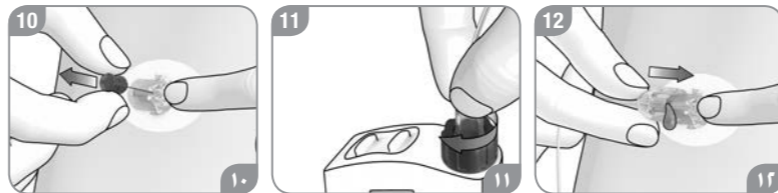
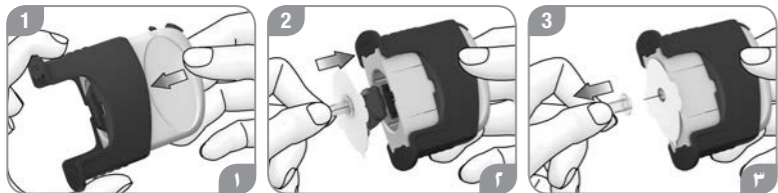
Insertion device

ACCU-CHEK®

Last update: 2018-06


04818300001/J

Instructions for use	4	הוראות שימוש	131
Gebrauchsanweisung	12	Használati útmutató	132
Instructions d'utilisation	20	Navodila za uporabo	140
Istruzioni per l'uso	28	Instrucțiuni de utilizare	150
Instrucciones de uso	36	Инструкции по использованию	158
Instruções de utilização	44	Інструкція з використання	166
Gebruiksaanwijzing	52	Kasutusjuhend	174
Brugsanvisning	60	Lietošanas pamācība	182
Bruksanvisning	68	Naudojimo taisyklės	190
Bruksanvisning	76	إرشادات الاستخدام	205
Käyttöohje	84		
Οδηγίες χρήσης	92		
Instrukcja obsługi	100		
Návod k použití	108		
Návod na použitie	116	US Warranty	206

Instructions for use	4	en
Gebrauchsanweisung	12	de
Instructions d'utilisation	20	fr
Istruzioni per l'uso	28	it
Instrucciones de uso	36	es
Instruções de utilização	44	pt
Gebruiksaanwijzing	52	nl
Brugsanvisning	60	da
Bruksanvisning	68	no
Bruksanvisning	76	sv
Käyttöohje	84	fi
Οδηγίες χρήσης	92	el
Instrukcja obsługi	100	pl
Návod k použití	108	cs
Návod na použitie	116	sk
שומיש תוארה	131	he
Használati útmutató	132	hu
Navodila za uporabo	140	sl

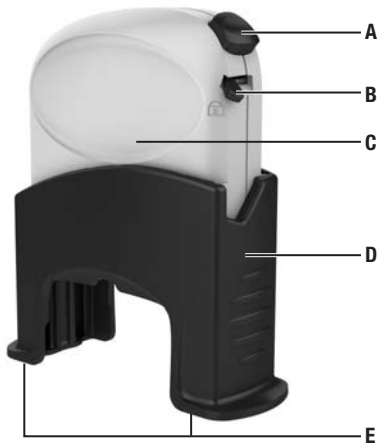Accu-Chek LinkAssist insertion device


Accu-Chek FlexLink infusion set (USA/Canada: Accu-Chek Ultraflex infusion set)

Head set

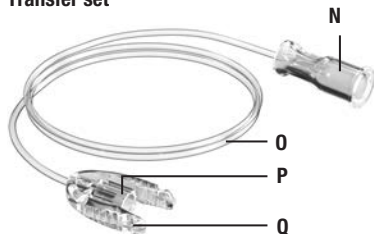Transfer set

Overview

Insertion device

- A** Release button
- B** Safety catch
- C** Pretensioning element
- D** Base
- E** Contact areas

Head set

- F** Handling aid
- G** Side wings
- H** Adhesive pad
- I** Protective film
- J** Cannula
- K** Introducer needle
- L** Needle cover
- M** Cannula casing

Transfer set

- N** Luer-lock connector
- O** Tubing
- P** Connector
- Q** Side wings

Intended use

The Accu-Chek LinkAssist insertion device is used to insert the Accu-Chek FlexLink infusion set (USA/Canada: Accu-Chek Ultraflex infusion set) under the skin.

These instructions for use feature the following symbol:

This symbol draws your attention to a warning. A warning must be heeded because it indicates a risk of injury or of damage to your health or the health of others. Not heeding the warnings can lead to life-threatening situations.

Warnings

- Read these instructions for use carefully before using the insertion device.
- Also read the instructions for use of the Accu-Chek FlexLink infusion set (USA/Canada: Accu-Chek Ultraflex infusion set).
- Dispose of the used product according to local regulations.
- The insertion device may only be used on one person.

Attaching the infusion set

- ▶ Choose an infusion site away from your waistline, any scar tissue, bones, navel and recent infusion sites.
- ▶ Wash your hands.
- ▶ Disinfect the chosen infusion site as instructed by your healthcare team.
- ▶ Let your skin dry completely.

1

- ▶ Push the pretensioning element of the insertion device into the base.

2

- ▶ Remove the head set of the infusion set from the packaging.
- ▶ Hold the head set by the needle cover.
- ▶ Insert the head set, with the blue handling aid pointing upwards, into the insertion device, until you feel the head set click into place.

Do not look into the opening of the insertion device when there is an infusion set in there.

3

- ▶ Remove the needle cover.

To avoid contamination or injury, do not touch the introducer needle or cannula after removing the needle cover.

- ▶ Keep the needle cover. You will need it later to safely dispose of the introducer needle.

4

Tip: The small tab on the protective film of the adhesive pad indicates the side to which you will later connect the transfer set. See illustration **A**.

- ▶ Remove both parts of the protective film from the adhesive pad.

5

- ▶ Pull the pretensioning element upwards, away from the base, as far as it will go.

Do not slide the safety catch to the  position, otherwise the insertion device will no longer be locked.

6

- ▶ Place the insertion device onto the chosen infusion site, holding it horizontally. It must be positioned in such a way that the whole surface of each contact area on the base of the insertion device is resting on the skin.

Tip: You should be able to comfortably reach the side which had the small tab on the protective film. That is where you will later connect the transfer set.

- ▶ Slide the safety catch to the  position. The insertion device is now unlocked.

7

- ▶ Press the release button. The cannula is inserted under the skin.
- ▶ Remove the insertion device.

- If the head set of the infusion set is still in the insertion device, push the pretensioning element back into the base. Continue with step **5**.

- If the cannula is not properly positioned under the skin, remove the adhesive pad from the skin and pull out the cannula. Start at step **1** with a new head set.

8

- ▶ Press the adhesive pad onto the skin.

9

- ▶ Hold the adhesive pad in place.
- ▶ Press the side wings of the handling aid together.
- ▶ Remove the handling aid in the direction of the arrow.

10

- ▶ Hold the cannula casing in place.
- ▶ Remove the introducer needle straight upwards.
- ▶ Dispose of the introducer needle in such a way that it will not cause injury to anyone. Place the introducer needle back into the needle cover or into a cannula disposal container.

11

- ▶ Screw the luer-lock connector into the insulin pump adapter.

Do not use any tools to screw the luer-lock connector into the insulin pump adapter. This could damage the luer-lock connector.

12

- ▶ Fill the transfer set as described in the User's Manual of the insulin pump until insulin drips from the needle inside the connector. There must not be any air bubbles in the cartridge or transfer set.

You must fill up the transfer set completely before you connect it to the cannula. The transfer set must not contain any air bubbles.

- ▶ Hold the cannula casing in place.
- ▶ Push the connector onto the cannula casing. You must hear and feel the connector click into place.

If you do not hear and feel the connector click into place, replace the head set and the transfer set.

- ▶ Fill the cannula casing with a U100 insulin bolus of 1.0 U (10 µL).

If you do not fill the cannula casing with an insulin bolus, you will not receive enough insulin.

Cleaning

- ▶ Moisten a cloth with water, with a mild hand soap or with a standard disinfectant (for example, 70 % ethanol).
- ▶ Wipe the outside of the insertion device. Do not dip the insertion device into the cleaning fluids.
- ▶ Allow the insertion device to dry completely.

Storage

Store the insertion device with the pretensioning element retracted and without tension. See illustration **B**.

- ▶ Pull the pretensioning element upwards, away from the base, as far as it will go.
- ▶ Slide the safety catch to the  position.
- ▶ Press the release button.

Guarantee

There is a guarantee of 18 months on the Accu-Chek LinkAssist insertion device for materials and workmanship, from the date of purchase.

US

For detailed warranty and service information please refer to the Warranty section at the end of this manual.

Customer support

In US, distributed by:

Roche Diabetes Care, Inc.
Indianapolis, IN 46256, **USA**
Toll Free: 1-800-280-7801
Accu-Chek Customer Care Service Center:
1-800-688-4578
www.accu-chek.com

Canada

Accu-Chek Pump Support: 1-800-688-4578
(toll free)
www.accu-chek.ca

Australia

Accu-Chek Enquiry Line: 1800 251 816
Pump Support: 1800 633 457
www.accu-chek.com.au

New Zealand

Accu-Chek Enquiry Line: 0800 80 22 99
Pump Hotline: 0800 696 696
www.accu-chek.co.nz

Singapore

Accu-Chek ExtraCare line: 6272 9200
www.accu-chek.com.sg

Roche Diabetes Care South Africa (Pty) Ltd.
Hertford Office Park, 90 Bekker Road
Vorna Valley, 1686

South Africa

Tel: +27 (11) 504 4600
www.accu-chek.co.za

Distributed in the United Kingdom by:

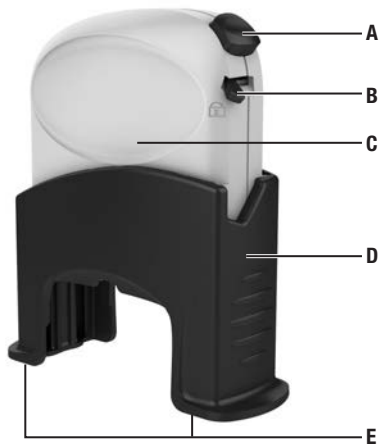
Roche Diabetes Care Limited
Charles Avenue, Burgess Hill
West Sussex, RH15 9RY, **United Kingdom**
Accu-Chek Pump Careline ¹⁾:

UK Freephone number: 0800 731 22 91
ROI Freephone number: 1 800 88 23 51

¹⁾ calls may be recorded for training purposes
Some mobile operators may charge for calls
to these numbers.

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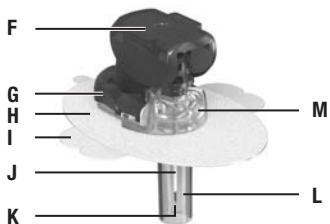
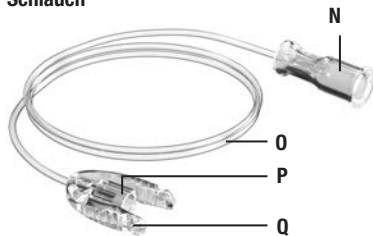
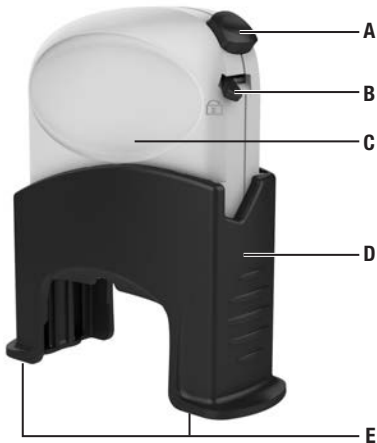
ar

Accu-Chek LinkAssist Limited 18-Month Warranty

Roche Diabetes Care, Inc. (“Roche”) warrants to the original purchaser of the Accu-Chek LinkAssist insertion device that your Accu-Chek LinkAssist insertion device will be free from defects in materials and workmanship for 18 months from the date of purchase. If, during this 18-month period, the insertion device does not work properly because of a defect in materials or workmanship, Roche will replace it with a new Accu-Chek LinkAssist insertion device or equivalent product free of charge. The warranty on the replacement insertion device will expire on the date of the original warranty expiration or 90 days after the shipment of a new insertion device, whichever period is longer. The purchaser's exclusive remedy with respect to the Accu-Chek LinkAssist insertion device shall be replacement.

This warranty does not apply to the performance of an insertion device that has been damaged by accident or has been altered, misused, tampered with, or abused in any way.

THE ABOVE WARRANTY IS EXCLUSIVE OF ALL OTHER WARRANTIES, AND ROCHE MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, THE IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE BE LIABLE TO THE PURCHASER OR ANY OTHER PERSON FOR ANY INCIDENTAL, CONSEQUENTIAL, INDIRECT, SPECIAL, OR PUNITIVE DAMAGES ARISING FROM OR IN ANY WAY CONNECTED WITH THE PURCHASE OR OPERATION OF THE ACCU-CHEK

LINKASSIST INSERTION DEVICE OR ITS PARTS. NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IF ANY IS IMPLIED FROM THE SALE OF THE INSERTION DEVICE, SHALL EXTEND FOR A LONGER DURATION THAN EIGHTEEN MONTHS FROM THE DATE OF PURCHASE.

Some states do not allow limitations on how long an implied warranty will last or the exclusion of incidental or consequential damages, so the above limitation and exclusion may not apply to you. This warranty gives you specific legal rights, which vary from state to state.

Warranty and Service Instructions

All requests for return of the Accu-Chek LinkAssist insertion device under the above warranty must be made to Roche Diabetes Care, Inc. Indianapolis, IN 46256, USA. You will be mailed a return authorization label, which must be affixed to your carton for shipping the system to Roche. Cartons received without this label will be returned to you at your expense.

Caution, refer to safety-related notes in the instructions for use accompanying this product.

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